

# Product information

Information about other products is available at: [www.demeditec.com](http://www.demeditec.com)



## Instruction for use

# ENA-4 Profile



**REF** DE7400



**4x 24 Tests**

**Immunometric Enzyme Immunoassay for  
the semi-quantitative determination of anti-  
ENA Antibodies**

## CONTENTS

<i>CONTENTS</i> .....	2
<i>NAME AND INTENDED USE</i> .....	3
<i>SUMMARY AND EXPLANATION OF THE TEST</i> .....	3
<i>PRINCIPLE OF THE TEST</i> .....	4
<i>WARNINGS AND PRECAUTIONS</i> .....	4
<i>CONTENTS OF THE KIT</i> .....	5
<i>STORAGE AND STABILITY</i> .....	5
<i>MATERIALS REQUIRED</i> .....	6
<i>SPECIMEN COLLECTION, STORAGE AND HANDLING</i> .....	6
<i>PROCEDURAL NOTES</i> .....	6
<i>PREPARATION OF REAGENTS</i> .....	7
<i>TEST PROCEDURE</i> .....	7
<i>INTERPRETATION OF RESULTS</i> .....	8
<i>PERFORMANCE CHARACTERISTICS</i> .....	9
<i>LIMITATIONS OF PROCEDURE</i> .....	9
<i>INTERFERING SUBSTANCES</i> .....	9
<i>REFERENCES</i> .....	9
<i>INCUBATION SCHEME</i> .....	10
<i>SYMBOLS USED WITH DEMEDITEC ASSAYS</i> .....	11

## NAME AND INTENDED USE

ENA-4-profile is an indirect solid phase enzyme immunometric assay (ELISA). It is designed for semi-quantitative determination of IgG class autoantibodies directed against the extractable nuclear antigens SS-A (Ro), SS-B (La), Sm and RNP/Sm in human serum or plasma. The assay is intended for in vitro diagnostic use only as an aid in the diagnosis of certain systemic rheumatic diseases.

## SUMMARY AND EXPLANATION OF THE TEST

Rheumatoid autoimmune diseases are often associated with the occurrence of autoantibodies against several nuclear or cytoplasmatic antigens. These so-called anti nuclear antigens (ANA) can be divided into three groups:

1. true anti nuclear antigens (ANA): directed against dsDNA, ssDNA, histone, nucleolic RNA and DNP
2. extractable nuclear antigens: Sm (Smith), n-RNP, Scl 70, Jo-1 and PM-1
3. cytoplasmatic antigens: SS-A (Ro) and SS-B (La)

In patients with Sjögren-Syndrome antibodies against the two cytoplasmatic antigens often occur in combination. Due to their strong association of SS-A and SS-B antibodies to the HLA-DR3 and DR2 phenotypes a genetic predisposition is suspected. The anti SS-A protein passes the placenta and may cause the development of SLE in neonates.

Immunoreactive proteins may occur in various combinations and also bound to 'host proteins' of viral origin. They induce synthesis of polyclonal autoantibodies, of the IgG, IgM and IgA class of immunoglobulins. Especially for mixed connective tissue diseases a relation to viral infections by EBV (Eppstein-Barr-Virus) is indicated.

Each class of immunoglobulins causes specific immune fluorescent pattern. Basically immunofluorescence titres correlate with the quantitative determination of IgG antibodies but the concentrations may considerably vary within each titre. Quantitation of IgG class antibodies extensively correlate with the diseases' activity. This makes it superior to immunofluorescence using HEp 2 cells, which may give variable results depending on their degree of activity. Also IF with *Crithidia luciliae* sometimes gives discrepant results.

Today the best investigated immunoreactive antigens are double-stranded DNA (dsDNA), single stranded DNA (ssDNA), Sm (Smith), sn-RNP (small nuclear ribonucleoprotein particles), the complex RNP/Sm which is stabilised by ribonucleic acid as well as SS-A (Ro) and SS-B (La). The antigen Scl 70, a 70 kD molecular weight protein is associated with scleroderma.

In rheumatoid autoimmune diseases various profiles of autoantibodies to these antigens can be detected. In a high incidence they are related to active and inactive systemic Lupus erythrematodes, mixed connective tissue diseases (Sharp Syndrome), rheumatoid arthritis, Sjögren-Syndrome, Scleroderma, photosensitive dermatitis and drug-induced lupus.

In Lupus patients typically anti-dsDNA antibodies can be detected. Patients without these antibodies very often show anti-ssDNA antibodies and anti-SS-A and anti-SS-B are present. A strong correlation between antibody concentration and severity of the disease has been observed with higher antibody concentrations in active phases of the disease. Thus quantitation is more informative compared to simple titering by immunofluorescence.

Measurement of anti-ssDNA provides additional information regarding antibody specificity and activity. Except in chronic inflammatory processes anti-ssDNA antibodies are not found in healthy subjects.

Most of these parameters are not specific for just one disease but they occur in various combinations. The pattern of different antibody combinations and their concentration together with

the whole clinical picture of the patient are helpful diagnostic tools in the assessment of rheumatoid autoimmune diseases.

The following graph gives brief information on the complexity of autoimmune diseases, occurring antibodies. It is not designed as a diagnostic schedule or program for ongoing diagnostic profiles:

Diseases	dsDNA	ssDNA	Histone	SS-A (Ro)	Anti- SS-B (La)	Sm	RNP/ Sm	Scl 70	Jo-1
Systemic Lupus erythematodes (SLE)	xxx	xx		x	x	xx	x		
Drug induced Lupus (LE)		xx	xxx						
Sharp-Syndrome / Mixed connective tissue diseases	x	x		x	x	xx	xxx		
Rheumatoid Arthritis	x	xx	x						
Sjögren-Syndrome	x	x		xx	xx				
Scleroderma	x		x			x		xx	x
Photosensitive Dermatitis Dermatomyositis	xx			x					xx

## PRINCIPLE OF THE TEST

Highly purified antigens SS-A (Ro), SS-B (La), Sm and RNP/Sm is bound to microwells. Antibodies to this antigen, if present in diluted serum, bind in the microwells. Washing of the microwells removes unbound serum antibodies. Horseradish peroxidase (HRP) conjugated anti-human IgG-IgG immunologically bind to the bound patient antibodies forming a conjugate/antibody/antigen complex. Washing of the microwells removes unbound conjugate. An enzyme substrate in the presence of bound conjugate hydrolyzes to form a blue color. The addition of an acid stops the reaction forming a yellow end-product. The intensity of this yellow color is measured photometrically at 450 nm. The amount of colour is directly proportional to the concentration of IgG antibodies present in the original sample.

## WARNINGS AND PRECAUTIONS

1. All reagents of this kit are strictly intended for in vitro diagnostic use only.
2. Do not interchange kit components from different lots.
3. Components containing human serum were tested and found negative for HBsAg and HIV by FDA approved methods. No test can guarantee the absence of HBsAg or HIV, and so all human serum based reagents in this kit must be handled as though capable of transmitting infection.
4. Avoid contact with the TMB (3,3',5,5'-Tetramethyl-benzidine). If TMB comes into contact with skin, wash thoroughly with water and soap.
5. Avoid contact with the Stop Solution which contains hydrochloric acid (1 M). If it comes into contact with skin, wash thoroughly with water and seek medical attention.
6. Some kit components (i.e. Controls, Sample buffer and Buffered Wash Solution) contain Sodium Azide as preservative. Sodium Azide ( $\text{NaN}_3$ ) is highly toxic and reactive in pure form. At the product concentrations, though not hazardous. Despite the classification as non-hazardous, we strongly recommend using prudent laboratory practices (see 8., 9., 10.)

7. Some kit components contain Proclin 300 as preservative. When disposing reagents containing Proclin 300, flush drains with copious amounts of water to dilute the components below active levels.
8. Wear disposable gloves while handling specimens or kit reagents and wash hands thoroughly afterwards.
9. Do not pipette by mouth.
10. Do not Eat, Drink, Smoke or Apply Makeup in areas where specimens or kit reagents are handled.
11. Avoid contact between the buffered Peroxide Solution and easily oxidized materials; extreme temperature may initiate spontaneous combustion.

Observe the guidelines for performing quality control in medical laboratories by assaying controls and/or pooled sera. During handling of all kit reagents, controls and serum samples observe the existing legal regulations.

## CONTENTS OF THE KIT

Package size            4 x 24 determ.

Qty.1	Divisible microplate consisting of 12 modules of 8 wells each, coated with individual highly purified SS-A, SS-B, Sm and RNP/Sm. Ready to use. The modules are colour coded: <table border="0" style="margin-left: 20px;"> <tr> <td>SS-A</td> <td>red</td> </tr> <tr> <td>SS-B</td> <td>blue</td> </tr> <tr> <td>Sm</td> <td>yellow</td> </tr> <tr> <td>RNP/Sm</td> <td>green</td> </tr> </table>	SS-A	red	SS-B	blue	Sm	yellow	RNP/Sm	green
SS-A	red								
SS-B	blue								
Sm	yellow								
RNP/Sm	green								
	Anti-ENA controls in a serum/buffer matrix (PBS, NaN <sub>3</sub> < 0,1 (w/w)). Ready to use.								
1 vials, 2,0 ml	Positive Control (1) (100 U/ml) .								
1 vials, 2,0 ml	Negative (2) (0 U/ml) .								
4 vials, 1,0 ml	Cut-off Calibrators (25 U/ml), specific for each individual antigen.								
1 vial, 20 ml	Sample buffer (Tris, NaN <sub>3</sub> <0,1% (w/w)), yellow, concentrate (5x).								
1 vial, 15 ml	Enzyme conjugate solution (PBS, PROCLIN 300 <0,5% (v/v)), (light red) containing polyclonal rabbit anti-human IgG; labelled with horseradish peroxidase. Ready to use.								
1 vial, 15 ml	TMB substrate solution. Ready to use.								
1 vial, 15 ml	Stop solution (1 M hydrochloric acid). Ready to use.								
1 vial, 20 ml	Wash solution (PBS, NaN <sub>3</sub> <0,1% (w/w)), concentrate (50x).								

## STORAGE AND STABILITY

1. Store the kit at 2-8°C.
2. Keep microplate wells sealed in a dry bag with desiccants.
3. The reagents are stable until expiration of the kit.
4. Do not expose test reagents to heat, sun or strong light during storage and usage.
5. Diluted sample buffer and wash buffer are stable for at least 30 days when stored at 2-8°C.

## MATERIALS REQUIRED

### Equipment

- Microplate reader capable of endpoint measurements at 450 nm
- Multi-Channel Dispenser or repeatable pipet for 100 µl
- Vortex mixer
- Pipets for 10 µl, 100 µl and 1000 µl
- Laboratory timing device
- data reduction software

### Preparation of reagents

- distilled or deionized water
- graduated cylinder for 100 and 1000 ml
- plastic container for storage of the wash solution

## SPECIMEN COLLECTION, STORAGE AND HANDLING

1. Collect whole blood specimens using acceptable medical techniques to avoid hemolysis.
2. Allow blood to clot and separate the serum by centrifugation.
3. Test serum should be clear and non-hemolyzed. Contamination by hemolysis or lipemia is best avoided, but does not interfere with this assay.
4. Specimens may be refrigerated at 2-8°C for up to five days or stored at -20°C up to six months.
5. Avoid repetitive freezing and thawing of serum samples. This may result in variable loss of autoantibody activity.
6. Testing of heat-inactivated sera is not recommended.

## PROCEDURAL NOTES

1. Do not use kit components beyond their expiration dates.
2. Do not interchange kit components from different lots.
3. All materials must be at room temperature (20-28°C).
4. Have all reagents and samples ready before start of the assay. Once started, the test must be performed without interruption to get the most reliable and consistent results.
5. Perform the assay steps only in the order indicated.
6. Always use fresh sample dilutions.
7. Pipette all reagents and samples into the bottom of the wells.
8. To avoid carryover contamination change the tip between samples and different kit controls.
9. It is important to wash microwells thoroughly and remove the last droplets of wash buffer to achieve best results.
10. All incubation steps must be accurately timed.
11. Control sera or pools should routinely be assayed as unknowns to check performance of the reagents and the assay.
12. Do not re-use microplate wells.

For all controls, the respective concentrations are provided on the labels of each vial. Using these concentrations a calibration curve may be calculated to read off the patient results semi-quantitatively.

## PREPARATION OF REAGENTS

### Preparation of sample buffer

Dilute the contents of each vial of the sample buffer concentrate (5x) with distilled or deionized water to a final volume of 100 ml prior to use. Store refrigerated: stable at 2-8°C for at least 30 days after preparation or until the expiration date printed on the label.

### Preparation of wash solution

Dilute the contents of each vial of the buffered wash solution concentrate (50x) with distilled or deionized water to a final volume of 1000 ml prior to use. Store refrigerated: stable at 2-8°C for at least 30 days after preparation or until the expiration date printed on the label.

### Sample preparation

Dilute all patient samples 1:100 with sample buffer before assay. Therefore combine 10 µl of sample with 990 µl of sample buffer in a polystyrene tube. Mix well. Controls are ready to use and need not be diluted.

## TEST PROCEDURE

1. Prepare a sufficient number of microplate modules to accommodate controls and prediluted patient samples.
2. Pipet **100 µl** of controls and prediluted patient samples into the wells.

	<i>SSA</i>	<i>SSB</i>	<i>S<sub>m</sub></i>	<i>FN<sub>P</sub>/S<sub>m</sub></i>
	1	2	3	4
<i>A</i>	NC	NC	NC	NC
<i>B</i>	CalC	CalC	CalC	CalC
<i>C</i>	PC	PC	PC	PC
<i>D</i>	P1	P1	P1	P1
<i>E</i>	P2	P2	P2	P2
<i>F</i>	P3	P3	P3	P3
<i>G</i>	P4	P4	P4	P4
<i>H</i>	P5	P5	P5	P5

NC Negative control  
CalC Calibration control  
PC Positive control  
P1, P2 Patient sample

3. Incubate for 30 minutes at room temperature (20-28°C)
4. Discard the contents of the microwells and wash 3 times with **300 µl** of wash solution.
5. Dispense **100 µl** of enzyme conjugate into each well
6. Incubate for 15 minutes at room temperature
7. Discard the contents of the microwells and wash 3 times with 300 µl of wash solution
8. Dispense **100 µl** of TMB substrate solution into each well
9. Incubate for 15 minutes at room temperature

10. Add **100 µl** of stop solution to each well of the modules and incubate for 5 minutes at room temperature
11. Read the optical density at 450 nm and calculate the results. Bi-chromatic measurement with a reference at 600-690 nm is recommended.

**The developed color is stable for at least 30 minutes. Read optical densities during this time.**

### Automation

The ENA-4-Profile ELISA is suitable for use on open automated ELISA processors. The test procedure detailed above is appropriate for use with or without automation.

## INTERPRETATION OF RESULTS

### Quality Control

This test is only valid if the optical density at 450 nm for Positive Control (1), Negative Control (2) and Cut-Off Calibrator complies with the respective range indicated on the Quality Control Certificate enclosed to each test kit! If any of these criteria is not met, the results are invalid and the test should be repeated.

### Calculation and interpretation of results

Evaluation of the ENA-4-Profile test is carried out by direct comparison of the optical density of each patient sample with the optical density of the cut-off calibrator.

Patient samples exhibiting optical densities higher than the optical density of the cut-off calibrator are considered to be positive.

Negative:	OD Patient	<	OD cut-off Calibrator
Positive:	OD Patient	>	OD cut-off Calibrator

For detailed semi-quantitative results, each patient-OD value can be expressed as an "Index Value". The Index Value is calculated by dividing the sample-OD by the Cut-off-OD.

$$\text{Indexvalue} = \frac{\text{OD}_{\text{Sample}}}{\text{OD}_{\text{Cut-off}}}$$

The calculation of Index Values is not influenced by variations of the sample-OD and/or Cut-off-OD. Index Values are recommended for long term validations (i.e. internal quality control samples).

Index Values are interpreted as follows: ENA-4-Profile ELISA

	(Index Value)
Negative:	< 1.0
Borderline:	1.0 -1.2
Positive:	> 1.2



## PERFORMANCE CHARACTERISTICS

### Specificity

The microplate modules are coated with the individual antigens SS-A (Ro), SS-B (La), Sm and RNP/Sm. All antigen preparations are highly purified by affinity chromatography. The ENA-4-Profile test is specific only for autoantibodies directed to these antigens. No cross-reactivities have been observed.

### Calibration

The assay system is calibrated against the internationally recognised reference sera from CDC, Atlanta USA, since no other international standards are available.

## LIMITATIONS OF PROCEDURE

The ENA-4-Profile ELISA is a diagnostic aid and by itself is not diagnostic. A definite clinical diagnosis should not be based on the results of a single test, but should be made by the physician after all clinical and laboratory findings have been evaluated.

## INTERFERING SUBSTANCES

No interference has been observed with haemolytic (up to 1000 mg/dL), lipemic (up to 3 g/dL triglycerides) or bilirubin (up to 40 mg/dL) containing sera. Nor have any interfering effects been observed with the use of anticoagulants. However for practical reasons it is recommended that grossly hemolyzed or lipemic samples be avoided.


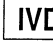



## REFERENCES

1. Antinuclear Antibody The Lancet, September 15, 1984: 611 - 13.
2. Froelich, Ch. J. et al. Clinical Value of an Integrated ELISA System for the Detection of 6 Autoantibodies. J. Rheumatol. 1990; 17: 2: 192 - 200.
3. Harley, J. B. and Gaither, K. K. Autoantibody. Rheum. Dis. Clin. 1988; Vol. 14, No. 1: 43 - 55.
4. Hartung, K. and Deicher, H. Systemischer Lupus erythematodes. Allergologie Nr. 7/1985; 8: 275 - 281.
5. Hartung, K. and Deicher, H. Symptome und Symptommuster beim systemischen Lupus erythematodes. Die Internistische Welt 1987; 9: 236 - 40.
6. Kühl, R., Sachs, V., Dörner, R., Utecht, B. und Gross, W. Ein Enzymimmunoassay als Suchtest für Antikörper gegen extrahierbare Kernantigene. Ärztl. Lab. 1987; 33: 5 - 9.
7. Reichlin, M. and Van Venrooij, W. J. Autoantibodies to the URNP particles: relationship to clinical diagnosis and nephritis. Clin. exp. Immunol. 1991; 83: 286 - 90.
8. Tzioufas, A. G. et al. Enzyme immunoassay for the detection of IgG and IgM anti-dsDNA antibodies: clinical significance and specificity. Clin. Exp. Rheumatol. 1987; 5: 247 - 253.

## INCUBATION SCHEME

- 1** Pipet **100 µl** control or patient sample  
→ Incubate for **30 minutes** at room temperature  
→ Discard the contents of the wells and wash 3 times with **300 µl** wash solution
- 2** Pipet **100 µl** enzyme conjugate  
→ Incubate for **15 minutes** at room temperature  
→ Discard the contents of the wells and wash 3 times with **300 µl** wash solution
- 3** Pipet **100 µl** substrate solution  
→ Incubate for **15 minutes** at room temperature
- 4** Add **100 µl** stop solution  
→ Leave untouched for **5 minutes**  
→ Read at **450 nm**

## SYMBOLS USED WITH DEMEDITEC ASSAYS

Symbol	English	Deutsch	Français	Español	Italiano
	Consult instructions for use	Gebrauchsanweisung beachten	Consulter les instructions d'utilisation	Consulte las instrucciones de uso	Consultare le istruzioni per l'uso
	European Conformity	CE-Konformitätskennzeichnung	Conformité aux normes européennes	Conformidad europea	Conformità europea
	In vitro diagnostic device	In-vitro-Diagnostikum	Usage Diagnostic in vitro	Para uso Diagnóstico in vitro	Per uso Diagnostica in vitro
	For research use only	Nur für Forschungszwecke	Seulement dans le cadre de recherches	Sólo para uso en investigación	Solo a scopo di ricerca
	Catalogue number	Katalog-Nr.	Numéro de catalogue	Número de catálogo	Numero di Catalogo
	Lot. No. / Batch code	Chargen-Nr.	Numéro de lot	Número de lote	Numero di lotto
	Contains sufficient for <n> tests/	Ausreichend für "n" Ansätze	Contenu suffisant pour "n" tests	Contenido suficiente para <n> ensayos	Contenuto sufficiente per "n" saggi
	Storage Temperature	Lagerungstemperatur	Température de conservation	Temperatura de conservación	Temperatura di conservazione
	Expiration Date	Mindesthaltbarkeitsdatum	Date limite d'utilisation	Fecha de caducidad	Data di scadenza
	Legal Manufacturer	Hersteller	Fabricant	Fabricante	Fabbricante
Distributed by	Distributor	Vertreiber	Distributeur	Distribuidor	Distributore
Content	Content	Inhalt	Conditionnement	Contenido	Contenuto
Volume/No.	Volume / No.	Volumen/Anzahl	Volume/Quantité	Volumen/Número	Volume/Quantità

Symbol	Portugues	Dansk	Svenska	Ελληνικά
	Consulte as instruções de utilização	Se brugsanvisning	Se bruksanvisningen	Εγχειρίδιο χρήστη
	Conformidade com as normas europeias	Europaeisk overensstemmelse	Europeisk överensstämmelse	Ευρωπαϊκή Συμμόρφωση
	Diagnóstico in vitro	In vitro diagnostik	Diagnostik in vitro	in vitro διαγνωστικό
				
	Catálogo n.º	Katalognummer	Katalog nummer	Αριθμός καταλόγου
	No do lote	Lot nummer	Batch-nummer	Αριθμός Παρτίδος
		Indeholder tilstrækkeligt til "n" test	Innehåller tillräckligt till "n" tester	Περιεχόμενο επαρκές για «n» εξετάσεις
	Temperatura de conservação	Opbevarings-temperatur	Förvaringstemperatur	Θερμοκρασία αποθήκευσης
	Prazo de validade	Udløbsdato	Bäst före datum	Ημερομηνία λήξης
	Fabricante	Producent	Tillverkare	Κατασκευαστής
Distributed by				
Content	Conteúdo	Indhold	Innehåll	Περιεχόμενο
Volume/No.	Volume/Número	Volumen/antal	Volym/antal	Όγκος/αριθ..